150mm

dobra

- Dispose preferably in special containers aimed for contaminated materials.
- Do not dispose without first isolate the needle with the cartridge case.
 - Since they are used, needles should not be re-sterilized or reused.
 - This product should be used by dentists only and just for the purposes described in these Instructions for Use.
 - Failure to observe such warnings is the responsibility of the professional.

Instructions for Use:

Turn ends of the capsule until the rupture of the seal. Do not use if seal is broken. Remove the smaller cover. Thread the needle on the syringe and remove the protection.

STERILE NEEDLE OF SINGLE USE. THE SEAL GUARANTEES THE STERILITY OF THE NEEDLE. /THE NEEDLES SHOULD BE DISPOSED AFTER USE. / RECYCLING IS FORBIDDEN.

Storage and Conservation:

Keep the product in its original package tightly closed and in a dry place. Under these conditions, the product is valid for up to 5 years as of the manufacturing date.

STERILE	R	
---------	---	---

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DENCOJET
Disposable gingival needle
English

PROFESSIONAL USE ONLY

Tri-beveled needle with thin walls which provide improved flow of the anesthetic solution. Sealed by melting weld and super safe protectors. Non pyrogenic. Sterilized by gamma radiation.

Composition:

Made of special high technology stainless steel. Siliconized to soften the penetration of the cannula into tissues.

Presentations:

- Box containing 100 sterilized disposable needles. / Size: 25mm x 0.3mm (30G) Short.
- Box containing 100 sterilized disposable needles. / Size: 16mm x 0.3mm (30G) X Short.
- Box containing 100 sterilized disposable needles. / Size: 38mm x 0.4mm (27G) Long.

Indications:

DENCOJET is indicated for injection of anesthetics according to the techniques indicated for that size of needle.

Precautions and Warnings:

- Do not use if package is damaged, open or wet.
- Do not use if seal is broken.
- During use, do not change needle direction while it is inside tissue.
- Never force the needle if there is any resistance.
- Do not fully insert the needle into the fabric. It is advisable to leave a remnant of 5mm or more of the metallic part visible in the oral cavity as a precaution in case of fracture of the needle. In this way, the fragment can be pinched for removal.
- Do not bend the needle. When the needle is folded, its structure may become impaired, making it more prone to fracture.
- The needles should be disposed and destroyed immediately after use to avoid injuries or reuse by non-authorized people.

dobrar