140mm

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Estrada do Guerenguê, 2059

Rio de Janeiro, RJ, Brasil

CEP: 22713-002

CNPJ: 33112665/0001-46

SAC: 0800 602 68 80

Fax: 55-21-3342-4009

www.novadfl.com.br - sac@dfl.com.br

**Indústria Brasileira/ Industria Brasileña/Made in Brazil**

2a dobra

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**PROFESSIONAL USE ONLY.**

**Composition:**

The red plate contains Brew, Copal mixture, Talc, Stearin, Red Iron Pigment and Macrolex 5-B Pigment. The green stick contains Brew, Copal mixture, Talc, Stearin, Green Chrome Oxide Pigment, Unilene BS-140 Resin and Unilene B-120 Resin.

**Indications:**

The red plates are indicated for general impressions.  
The green plates are indicated for additions and godiva impressions improvements, for peripheral sealing in total prosthesis, absolute isolation clamp stabilization, semi-adjustable articulators facial arch registration, stabilization of steel matrixes for extensive alloys and copper ring moldings for incrustations, veneers, etc.

**Presentations:**

Godiva Plates Box with 8 red plates.  
Godiva Sticks Box with 15 green sticks.

**Storage and Conservation:**

**GODIVA** must be stored in its original package and in a dry place. Must be kept away from excessive heat ( avoid temperatures higher than 40° C ) Under these conditions, the product is valid for up to 5 years after manufacturing date. Do not use this product if shelf life had expired.

**Instructions for use:**

The use of gloves is recommended during the procedure. Soften the red plates in warm water at 52°C. The green sticks may be flame-molded, taking care not to burn the material. Manipulate with your fingers into a homogenous mass. Adjust the previously selected mold. Flambé the surface of the material with an alcohol torch. Dip quickly into the warm water. Place in the mouth, taking the impression which may be removed after the material has hardened (approximately 3 minutes). In order to speed up the hardening, spray the surface of the mold with a jet of cold water. Remove the mold only after it has completely hardened. If the model remains unfinished, store the mold in water in a container away from heat and dust until further use.

After use or handling disinfect material with alcohol 70 vol.

**Precautions:**

Avoid the use of **GODIVA** in patients with previous history of allergy. In case of allergy, you must discontinue the use immediately and advise the patient to see a doctor. This product must be used only by dentists and for the intended use described in the Instructions for Use. Product wastes must not be reused. They must be disposed as the local law in force.

2a dobra

1a dobra