03972 Placa Base - Instrucao de Uso

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2a. dobra

PLACA BASE
Base plate for Prosthesis
English

PROFESSIONAL USE ONLY

Composition:

Brown SUPPORT PLATE contains: Copal/Stearin mixture, Talc, Shellac, Stearin and Calcium Stearate.

Pink SUPPORT PLATE contains: Copal/Stearin mixture, Talc, Shellac, Stearin, Calcium Stearate, Titanium Oxide and Macrolex Red Dye.

Indications:

It is indicated as support for prosthesis making.

Presentations:

Cardboard-box containing 25 superior or inferior thick, brown and pink plates.

Cardboard-box containing 50 superior or inferior thin, brown and pink plates.

1a. dobra

Storage and Conservation:

SUPPORT PLATE should be stored at room temperature (between 15°C and 30°C). After use, keep packages well closed. Do not expose to high temperatures. Since stored at these conditions, the product is valid for up to 5 years after manufacturing. Do not use expired products.

Way to use:

The use of glasses and protection gloves is recommended during the procedure.

Put the plate in a warm water bath.

Attention: Temperature should not exceed 60°C during procedure.

When the plate becomes soften, apply over a dry gypsum model.

Mould the plate with your fingers, previously wet in cold water. Cut excesses and let it cool. The plate will thus be ready for the next steps, such as wax flat and teeth molding.

Product wastes must not be reused. They must be disposed as the local law in force.

Precautions:

Avoid the use of this product in patients with a history of allergy to anyone of the components present in the formulation. In case of allergy, immediately suspend the use and advise your patient to seek for a doctor.

This product should be used just by dentist or prosthodontist and just for the purposes described in these Instructions for Use.

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