01167 Natural Shade - Instr de Uso PEI

CE
2460

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English
NATURAL SHADE
Light-cured Micro hybrid Restorative Composite

PROFESSIONAL USE ONLY

NATURAL SHADE is a light-cured micro hybrid restorative resin, radiopaque with excellent consistency. NATURAL SHADE is manufactured with nanotechnology, under strict quality control, which provides superior esthetic and functional results to the professionals.

Composition:

NATURAL SHADE contains:
Bisphenol A Diglycidylmethacrylate (Bis-GMA), Triethyleneglycol Dimethacrylate (TEGDMA), Urethan dimatacrylate (UDMA), Bisphenol Glycidyl methacrylate (Bis-EMA), Aluminum Borosilicate, Silica, Camphoroquinone, Butylated hydroxytoluene (BHT) and Pigments.

Presentations:

- Envelope containing 1 syringe with 4g of NATURAL SHADE + 1 bottle with 4mL of dentin adhesive NATURAL BOND DE PLUS.
- Kit containing 4 syringes with 4g of NATURAL SHADE + 1 bottle with 4mL of dentin adhesive NATURAL BOND DE PLUS + 1 syringe with 3g of acid conditioner ALPHA ETCH.
- Kit containing 5 syringes with 4g of NATURAL SHADE + 1 bottle with 4mL of dentin adhesive NATURAL BOND DE PLUS + 1 syringe with 3g of acid conditioner ALPHA ETCH.
- Kit containing 7 syringes with 4g of NATURAL SHADE + 1 bottle with 4mL of dentin adhesive NATURAL BOND DE PLUS + 1 syringe with 3g of acid conditioner ALPHA ETCH.
- Refil containing 1 syringe with 4g of NATURAL SHADE.
- Refil containing 2 syringes with 4g of NATURAL SHADE.
- Refil containing 3 syringes with 4g of NATURAL SHADE.
- Refil containing 4 syringes with 4g of NATURAL SHADE.
- Refil containing 5 syringes with 4g of NATURAL SHADE.

NATURAL SHADE is available in the following shades:

- Enamel A1; A2; A3; A3,5; A4; B1; B2; B3; C2; C3; C4; D3;
- Enamel Bleach CL;
- Dentin A2; A3; A3,5; B2; C2; UNIVERSAL;
- Incisal

Indications:

NATURAL SHADE is indicated for restorations classes I, II, III, IV and V; Restores direct veneers, diastemas closure, core preparation, tooth fragment bonding and periodontal splinting with fibers.

Contraindication:

NATURAL SHADE is contraindicated in patients with acrylates sensitivity history or some other component of the formulation. If allergic reaction occurs, seek medical assistance, remove the product and discontinue its use if necessary.

Warnings:

Use gloves when handling the NATURAL SHADE. Avoid prolonged contact with soft tissue or skin. In case of contact, immediately remove the material and rinse with water. Avoid eye contact. In case of contact, rinse immediately with water and seek medical assistance. Avoid the use of eugenol-based materials because they inhibit the polymerization of the NATURAL SHADE. After opening, the product should be stored in its original container tightly closed and under the conditions described in "Shelf Life and Storage", otherwise the product will lose its features. Rests or waste of material used lose their features and therefore should not be reused, discard them in accordance with legislation in force. Keep out of the reach of children. NATURAL SHADE should be used only by dentists and for the intended use described in the product Instructions for Use.

Instructions for use:

- Remove completely the restoration and/or the caried tissue. If necessary, bevel the enamel margins;
- Cleanse the tooth to be treated and the adjacent teeth as well;
- Select the shades, ideally in broad day light;
- Isolate the cavity;
- If necessary, proceed with pulp protection by using a calcium hydroxide cement and/or Glass Ionomer VITRO FIL LC;
- Acid etch using ALPHA ACID or ALPHA ETCH, by 30 seconds in enamel and 15 seconds in the dentin;
- Rinse with water spray and remove the excess moisture avoiding dehydrating the dentin;
- If necessary, apply a transparent matrix band by using interdental wedges;
- Apply a layer of dentin adhesive NATURAL BOND DE, rubbing by 5 seconds. Wait for 20 seconds and apply gently a jet of air by 5 to 10 seconds;
- Apply a second layer of NATURAL BOND DE. After waiting for 20 seconds, light cure the adhesive by 20 seconds;
- Dose a portion of NATURAL SHADE directly onto the mixing pad. Apply NATURAL SHADE in the cavity in increments up to 2mm;
- Light cure each increment for 20 seconds, finalizing with a light cure of 40 seconds;
- Do the finishing in three stages:
 1. Finishing with diamond burs;
 2. Polishing with NATURAL POLUX siliconized polishers;
 3. Application of the glazer or surface sealant NATURAL GLAZE.

Shelf Life and Storage:

NATURAL SHADE must be stored between 2 and 30°C. NATURAL SHADE fulfills the intended purpose until 3 years after manufacturing date. Do not use this product if shelf life has expired. Product wastes must not be reused. They must be disposed as the local law in force.

Technical Data According to ISO 4049:

Average particle size:0.7 µm
Range of particle size:0.04-2.8 µm
Percentage by volume of total inorganic filler:60-64 %
Percentage by weight of total inorganic filler:75-79 %

EC REP Nova DFL Industrie et Commerce France
10-12 Boulevard Vivier Merle,
Tour Oxygène
Lyon Cedex 03

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DFL Indústria e Comércio S.A.