04651 Natural Bond DE Plus - Instrucao de Uso PEI

CE
2460

Português

NATURAL BOND DE PLUS
Adesivo Monocomponente

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NATURAL BOND DE PLUS
Single Component Adhesive

PROFESSIONAL USE ONLY

NATURAL BOND DE PLUS is a single component light cured bonding agent for enamel and dentin.

NATURAL BOND DE PLUS promotes adhesion in direct and indirect procedures and is compatible with all materials for dental restoration, as conditioners, composite and resin cements. It contains fluoride.

Composition:

NATURAL BOND DE PLUS contains: PMGDM, 2 HEMA, TEGDMA, Canphoroquinone, 4 EDAMB, PHFA, 2,6 DI, Bis-GMA, Neutral Hydrated Alcohol.

ALPHA ETCH contains: Phosphoric Acid, Aerosil 200, Methylene Blue CI52015, Bi distilled Glycerin y Deionized Water.

Presentations:

- Kit containing 2 bottles (4 mL, each) of NATURAL BOND DE PLUS + 1 syringe 2.5 mL of ALPHA ETCH + accessories (applicator tips).
- Kit containing 2 bottles (4 mL, each) of NATURAL BOND DE PLUS.
- Package containing 1 bottle (4 mL) of de NATURAL BOND DE PLUS (For replacement).

Indications:

NATURAL BOND DE PLUS is indicated for adhesion by any composite and compomer to enamel or dentin, adhesive cementation of indirect restorations type veneers, inlays and ceramic crowns, ceromer and composite (since associated with a resin cement).

Contraindication:

NATURAL BOND DE PLUS is contraindicated in patients with previous history of allergy to compounds containing acrylate. In the case of allergy, you must discontinue the use immediately and advise the patient to see a doctor.

Warnings:

- Use gloves when handling the NATURAL BOND DE PLUS.
- Avoid contact with soft tissue or skin. In case of contact, immediately remove the material with cotton soaked with water and rinse thoroughly with water.
- Avoid eye contact. In case of contact, rinse immediately with water and seek medical assistance.
- Avoid the use of eugenol-based materials because they inhibit the polymerization of the NATURAL BOND DE PLUS.
- In deep cavities, perform protection of pulp-dentin complex.
- Use, whenever possible, absolute isolation.
- After opening, the product should be stored in its original container tightly closed and under the conditions described in "Shelf Life and Storage", otherwise the product will lose its features.
- Rests or waste of material used lose their features and therefore should not be reused, discard them in accordance with legislation in force.
- Keep out of the reach of children.
- NATURAL BOND DE PLUS should be used only by dentists and for the intended use described in the product Instructions for Use.

Direct Bonding of Composite to Dentine/Enamel Procedures:

- 1 - Isolate, clean and prepare the tooth for a restoration.
- 2 - Etch dentine for 30 seconds and enamel for 15 seconds with Alpha Etch.
- 3 - Rinse with water spray.
- 4 - Remove the excess of water with a jet of air, avoiding dentine dehydration. Moisture should be visible on the surface (shiny dentine), but should not form puddles, which should be removed with brush or absorbent paper. Avoid saliva contamination.
- 5 - Place 2 drops of NATURAL BOND DE PLUS in a mixing tray or a Dappen pot. Place the first coat, scrub for 5 seconds, wait 20 seconds, gently apply an air jet for 5 to 10 seconds to remove the solvent and then apply a second coat. Avoid too thin coats, generated by the use of excessive air. Wait 20 seconds.
- 6 - Light cure for 20 seconds. The surface should appear glossy; if not, repeat the procedure.
- 7 - Proceed with the placement of the increments of resin.

Indirect Restorations Procedures:

- 1 - After tooth preparation, perform the isolation and prophylaxis.
- 2 - Treat the piece internal surface according to the type of material.
- 3 - After tooth conditioning with Alpha Etch, apply the adhesive as steps 5 and 6 above.
- 4 - Proceed with the application of the resin cement selected and settlement of the workpiece.

Shelf life and Storage:

NATURAL BOND DE PLUS must be kept in dry place and avoid exposure to sunlight. Do not expose to excessive heat (temperatures higher than 40°C). After use, keep the bottles tightly closed. NATURAL BOND DE PLUS shelf life is 2 years after manufacturing date. Do not use this product if shelf life has expired.

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DFL Indústria e Comércio S.A.